for FlexCone and 2.3 mm for Mini FlexCone, giving excellent stability to the final prosthesis.

Method of placement: Take the component into the internal internal connection and give the squeeze by tapping it with the connection key for FlexCone with 60 Ncm torque and the connection key for Mini FlexCone with a torque of 50 Ncm. For the Frictional FlexCone line seat the component with a special key called Frictional Beater with 3 to 5 little tappings.

**ANGLED FLEXCONE PILLAR - GOLD PROSTHETIC LINE**

Description/Note: This is a definitive component intermediary which is seated on the implant internal connection (Internal Hexagon Morse, Hexagonal Cone Morse, Cone Morse and Frictional Cone Morse) by screw-retention. It is indicated for multiple screw-retained prostheses. They are presented in 1.5, 2.5 and 3.5 mm heights.

It features a hexagon with a height of 2.6 mm giving excellent stability to the final prosthesis.

**Method of placement:** Take the component into the internal connection of the implant and give tightening by tapping with the connection key for FlexCone with a torque of 60 Ncm and with key connection for Mini FlexCone with a torque of 50 Ncm. For FlexCone of Frictional line seat with a special key called Frictional Beater with 3 to 5 little tappings.

P.S.: The lab screws, transferent and final screws are equal to the Mini FlexCone. Only the healing abutment and the analog are special for the Angled FlexCone System. These components are fabricated with titanium grade V and receive special treatment resulting in gold color.

**HEALING CAPS, COVER SCREW and PROTECTION CYLINDER**

Description/Note: These are provisional prosthetic components to be used directly over the implant and intermediate gingival and trans-gingival components

to provide an esthetic contour favoring molding gum during the healing process. The sizes (height and diameter) of DSP Healing Caps should be chosen accordingly to the diameter of the prosthetic element of the region and the thickness of fibromucous site.

Method of placement: Take the component on the internal connection of the implant, and give the squeeze by tapping digital 1.2 key (healing caps and protection cylinders) and 0.9 digital key for implant cover screw. Apply the following torque: 30 Ncm for Healing Caps; 15 Ncm for implant cover screw and 10 Ncm for protection cylinders.

**MATERIALS MANUFACTURING**

The provisional and Definitive Prosthetic Pillars are made with titanium medical IV.

**PRESENTATION**

Are commercially sterile by gamma irradiation and packed in blister containing a unit product. Depending on the model a screw can be accompanied.

**RISK PREVENTION, PRECAUTIONS, RESTRICTIONS**

- Do not use the product if the package is damaged.
- Do not use this product if it has expired;
- Products must be used with surgical instruments, cleaned and sterilized;
- Sterile product for single use - FORBIDDEN REPROCESSING;
- Misuse, abuse or excessive force can cause stripping, breakage or irreversible damage to the product;
- The use of these products requires a specific professional expertise in the field of implant dentistry applied to the prosthesis, so the DSP Biomedical is not responsible for technical errors of misapplication or improper handling of the product component by the professional, improper storage or transport provided by third parties.
- Attach extra label that comes with the product registration in the patient form so if necessary, can be traced.

**PRODUCT DISPOSAL**

If there is the need to dispose of the product, the trader should follow the municipal health laws advocated by the Health Surveillance for contaminated products.

**WARNINGS**

The product is marketed sterile. It is not recommended to place it in an oven or autoclave. Special care with the instrumental employee: It is recommended to use filtered water or deionized water for washing the instruments before and after use. Rinse with clean cold water. Dry the product immediately after washing and cleaning check against white background. Do not use abrasives during washing. Use only mild detergent with a soft sponge.

**COUNTER-INDICATIONS:** The Definitive and provisional prosthetic DSP Bioemdcial should be used exclusively in OSTEOFIT System implants. The DSP Biomedical is not responsible for problems in prosthetic components used on other brands of implants. There is no contraindication as long as if the products were to be used for the purpose and directions suggested.

**ADDITIONAL INFORMATION TO PROFESSIONAL**

Advise the patient to correctly take prescribed medication. Do not chew on the operative area. Sanitize mouth with mouth wash and brushing with specific mouthwash.

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<b>ENGLISH</b>
----------------

**Trade Name:** TITANIUM PROSTHETIC PILLAR (TPP)
**Technical Name:** PROVISIONAL AND DEFINITIVE PROSTHETIC COMPONENT
**Definition:** INVASIVE DEVICE
Rule/Classification (as CE 93/42/ECC) 5/II-a

Note: The TPP was specially developed by the DSP Biomedical to be used directly on the line of OSTEOFIT implants for dental restoration in a prosthesis shape. Its function is to restore numerous models of TPP which should be used according to the clinical indicators for the installed implant techniques.

**ORING E MICRO ORING**

Descripción/Note: This definitive prosthetic component model is used between the implant and removable denture. It aims to retain the mobile prosthetic implantation.

They are threaded into the implant and on top of this component is a hexagon which serves only for torquing the component. On this connection there is a hexagon type ball for retaining the prosthesis. They are available in height of 1, 2, 3, 4 and 5 mm.

Method of placement: Must be placed with DSP Biomedical O’ring key and aid key torquimeter applying maximum torque of 45 Ncm. In total prosthesis should be created a space for placing the metal ring so you can plug in the O’ring or Micro O’ring. P.S. 1: In this type system “Male and Female”the denture is fixed to the Micro O’ring or mucosa. The attachment is type implant-mucosa supported. P.S. 2: Does not follow the metal o’ring and the protective capsule or transfer molding components.

**STRAIGHT, ANATOMICAL AND ANGLED SCREW-RETAINED ABUTMENTS**

Descriptions/Uses: They can be used for temporary or permanent prosthesis, fixed or screw-retained, pre-shaped, i.e. there is no possibility of making a casting or over-casting on. The Straight Abutment has a height of 1 mm only while the Anatomical and the titanium UCLA have heights 1, 2 and 3 mm.

Method of placement: Fit the component over the implant, place the 1.3 (squared) screw ant thread the screw into the implant. Apply a 45 Ncm torque. The locking screw will pass between the sleeve and the implant.

**CONICAL (TAPERED) PILLAR AND MINI PILLAR**

Description/Note: It is a component used exclusively as an intermediary between the implant and definitive or provisional prosthesis. It can be straight or angled (15 or 25 degrees) containing a hexagon (anti-rotational system) when there is an indication for unitary prosthesis or without hexagon (rotational system) when there is na indication for multiple prosthesis. The conical pillar and mini pillar are indicated for partial fixed denture in general, specifically in the esthetic zone containing between 3 mm (for pillar) and 2 mm (for mini pillar) gingival height. Indicated in cases where vertical space is always greater than 7 mm.

Method of placement: Thread the component over the implant with the DSP Biomedical key for conical pillar and mini pillar applying a torque of 20 Ncm.

Contra Indication: The conical pillar and mini pillar are not indicated in buccal or lingual angulations where the aesthetics is compromised; In angulations of implants greater than 30 degrees where the difficulty of insertion prevents the perfect seating of the prosthesis; Cases where there is less than 2 mm gingival sulcus; Where the outcome os a possible gum recession result in exposure of the metal collar that is aesthetically unacceptable.

**MULTIPLE FLEXCONE PILLAR AND MINI FLEXCONE**

Description/Note: This is a definitive intermediary component which seats on any implant with internal connection (Internal Hexagon Morse, Hexagonal Cone Morse, Cone Morse and Frictional Cone Morse) through threading only. It is indicated for unitary cemented prosthesis, screw-retained o O’ring. It is presented at the Heights of 0,3, 0,75, 1,5, 2,5 and 3.5 mm (family) and height of 1, 2 and 3 (family 2). It features a hexagon with a height of 2.6 mm

dobra...

dobra...

dobra...

dobra...

dobra...

dobra...