PORTUGUÊS

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ENGLISH

Trade Name: **CASTABLE PROSTHETIC PILLAR WITH METALLIC BASE**

Technical Name: **CASTABLE PROSTHETIC PILLAR WITH METALLIC BASE**

Definition: **NON INVASIVE DEVICE**

Rule / Classification (as CE Directive 93/42/ECC): 5/IIa

PRESENTATION

Are commercially no sterile and packed in blister containing a unit product. Depending on the model a screw can be accompanied.

Note: The Castable Prosthetic Pillar with Metallic Base was specially developed to be used as

a mold casting alloys. It comes standard for creating a screwed crown. This abutment, after casting, is very suitable for locations with minimum thickness of the soft tissue. Its plastic ensures the lab setting to cut the excesses and shape it to the desired form and its base metal allows the perfect fit with the implant. The Castable Prosthetic Pillar with Metallic Base models are available for: HE, HIM, CMH, CMI, FC, FCM

Technical Features: It has a cylindrical shape with perforation to access internal screw for prosthesis fixation. The Castable UCLA with metallic base is an intermediate component between the implant and prosthetic crown. In relation to implant external hex, the UCLA may be rotational or antirrotational in diameters regarding the implant occlusal platform.

PRODUCT COMPOSITION: are made of Polyacetal (Polyoxymethylene) and Cobalt-Chrome alloy band.

PRECAUTIONS, RESTRICTIONS AND WARNINGS USE BY PROFESSIONALS ONLY

. The making of prostheses on dental implants requires a specific professional specialization. It will be the dental surgeon's or the prosthodontist's responsibility to make sure they've been previously qualified to use this product.

. Careful clinical and radiographic evaluations are necessary for the correct planning of the treatment, which must take into consideration the most adequate prosthetic options regarding chewing force balance, occlusal adjustment, esthetics and other factors related to the prosthesis' good performance. The exchange of information between the surgeon, the prosthodontist, and the laboratory technician is critically important for the treatment's success.

. The product cannot be reutilized or reprocessed

. The maximum possible torque in the definitive detached bolts is of 20 Ncm.

. Do not use it if the packaging has been violated or if the validity term has expired.

. **NON-STERILE PRODUCTS.**

. The abusive use of alcoholic beverages, tobacco, drugs, corticoids, or a lack of adequate oral care can significantly hinder the treatment's success.

. We recommend that the identification adhesive labels that come with the product is annexed to the documentation to be delivered to the patient, to the clinical protocol and to the fiscal documentation that will generate a collection. Note: To have a safe combination, Abutments should be used in accordance with the connection and implant platform.

CARE WHEN DISCARDING THE PRODUCT

The product's disposal must comply with the applicable environmental and biosafety laws. Do not discard contaminated products in the general waste.

STORAGE AND TRANSPORT

This product should be kept in their original packaging, kept in a cool, protected from moisture and sunlight.

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