DSP *porcelain*

ETCH

- EN | ENGLISH
- PT | PORTUGUÊS
- ES | ESPAÑOL

DSP *white*

ENGLISH

To access the IFU go to dspbiomedical.com/ifu with code: 0.1200

TRADE NAME: DSP Porcelain Etch
TECHNICAL NAME: Conditioning Acid

Please read all information of this instruction for use carefully before applying the product.

INSTRUCTIONS FOR USE
PROFESSIONAL USE ONLY

Keep for consultation until the total consumption of the product.
Keep this instruction until there is no more interaction of the product with your patient.

CONTENT:

- 1 acid conditioner syringes containing 3g with 1 applicator tip,
- 1 instruction manual for the professional and patient.

PRODUCT DESCRIPTION:

DSP Porcelain Etch is a hydrofluoric acid-based gel in concentrations of 5 or 10%, for the acid conditioning of porcelains, with the objective of increasing the adhesion of resinous materials.

PRODUCT INDICATION:

Surface conditioning of dental porcelain for adhesion in the process of restorations, such as feldspathic porcelain, ceramic glass and fixations of orthodontic brackets only on porcelain teeth.

COMPOSITION:

Hydrofluoric Acid 5 or 10%, Dye, Thickeners and Deionized Water.

INSTRUCTIONS FOR USE:

- 1st Feldspathic Porcelain
Condition for 1-2 minutes with DSP Porcelain Etch 10% HF;
- 2nd Glass ceramic reinforced with Leucite
Condition for 1 minute with DSP Porcelain Etch 10% HF;
- 3rd Lithium Disilicate based glass ceramic
Condition for 20-30 seconds with DSP Porcelain Etch 5% HF;
- 4rd Zirconia
Non-conditionable.

PRECAUTIONS AND CONTRAINDICATIONS:

Hydrofluoric acid is corrosive. Avoid contact with enamel, dentin, eyes, skin and mucosa. It is recommended to use eye protection for the patient, dentist and assistant during its use. When applied to the patient's mouth, the oral tissues must be protected with Vaseline and, whenever possible, perform absolute isolation. In case of accidental contact with eyes, skin or mucosa, rinse thoroughly with water. If necessary, seek medical attention.

SIDE EFFECTS:

Contact with soft tissues and ingestion of the product causes irritation and fluoride poisoning.
Discontinue use of the product in case of allergic reactions.

CONSERVATION AND STORAGE:

- Keep the product in its packaging always closed.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0°C to 30°C.
- If they find malfunctions, request the replacement of the material.

DATE OF MANUFACTURE: PACKING

EXPIRATION DATE: 2 YEARS AFTER THE DATE OF MANUFACTURE.

WARNING:

Respect the expiration date. Never use the product if it has expired. Keep out of reach of children. For dental use only. Before using observe the appearance of the product, if you notice any anomalies, contact the manufacturer or a professional immediately. The manufacturer is not responsible for damages caused by other uses or by incorrect handling. This material has been manufactured for dental use only and must be handled according to the

instructions for use. It is the responsibility of the user to prove, before the use of this product, that this material is compatible and suitable for the desired use, especially when this use is not indicated in these instructions for use. Data descriptions do not constitute any kind of guarantee and, therefore, do not have any link. Do not reuse empty packages. Do not reuse empty packaging for other purposes. To dispose of the product, follow local legislation.

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Keep away from light
Mantener afastado da luz
Mantener alejado de la luz

PROIBIDO REPROCESSAR

Prohibited
Reprocess
Proibido
Reprocessar
Proibido
reprocessar

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