# DSP *white*

ENDO GEL

EN | ENGLISH

PT | PORTUGUÊS

ES | ESPAÑOL

DSP  
*white*

# DSP white

ENDO GEL

ENGLISH

To access the IFU go to  
dspbiomedical.com/ifu  
with code: 0.5001

TRADE NAME: DSP White Endo Gel  
TECHNICAL NAME: Dental Whitening Agents

Please read all information of this instruction for use carefully  
before applying the product.

DENTAL WHITENER FOR CLINICAL USE.  
FOR HOME APPLICATION UNDER THE SUPERVISION OF A DENTAL  
SURGEON  
SALE UNDER PRESCRIPTION (ACCORDING TO RDC NO. 6 OF  
2015)

Keep for consultation until the total consumption of the  
product.  
Keep this instruction until there is no more interaction of the  
product with your patient.

CONTENT:  
• 1 whitening gel syringe containing 3g with 10 applicator tips,  
• 1 instruction manual for the professional.

PRODUCT DESCRIPTION:  
DSP White Endo Gel is a 35% carbamide peroxide based  
gel that has been developed based on products already  
known worldwide and highly recommended for whitening  
non-vital teeth. It has a neutral pH for application inside the  
pulp chamber and removal of pigments that color the tooth  
internally and is already ready for application.

PRODUCT INDICATION:  
DSP White Endo Gel is a 35% Carbamide Peroxide-based  
whitening system for professional use, indicated for the  
whitening of non-vitalized teeth. DSP White Endo Gel should  
be administered inside the pulp chamber of the tooth to  
be whitened, and should remain for a period to oxidize the  
existing pigments.

COMPOSITION:  
Carbamide Peroxide, Carbomer, Humectant (Glycerol),  
neutralizers and Deionized Water.

INSTRUCTIONS FOR USE:  
1st step  
Verify through clinical and radiographic examination the  
level of excellence of endodontic filling;  
2nd step  
Open lingual access to the pulp chamber. Remove all filling  
material and gutta-percha 2 to 3 mm apical from the  
cement-enamel junction;  
3rd step  
Remove all fill material from the access opening to the  
level determined from the x-ray All restoration material,  
temporary and/or endodontic filling material must be  
removed to allow the whitening agent to contact the internal  
structure of the tooth, especially the vestibular face, to obtain  
optimal results.  
4th step  
Apply DSP White Endo Gel to the pulp chamber avoiding soft  
tissues. Place a small cotton ball on the gel. Leave enough  
space for temporary restoration;  
5th step  
Mix conventional glass ionomer with a measure of powder to  
a measure of liquid until a firm sealant is obtained.  
NOTE: It must be firm enough to roll between your fingers and  
thumb Apply a layer of 1 to 2 mm to ensure sealing.  
6th Step  
Apply temporary material to seal the access opening.  
Repeat the whitening procedure every 3 to 5 days, until the  
desired color change is obtained;  
7th Step  
Wait 8 to 10 days before restoring the tooth. This will allow the  
tooth tone to stabilize;  
8th Step  
Remove the temporary restoration material and the  
whitening material to the level of the glass ionomer sealant  
material, rinse and clean the access opening. Restore with  
definitive restoration procedures.

PRECAUTIONS AND CONTRAINDICATIONS:  
• This is a product that must always be indicated and  
supervised by a dentist;  
• The product should not be used by pregnant and/or  
lactating women and in patients under 15 years old;  
• Avoid the purposeful and excessive ingestion of the product;  
• Do not use in patients with any sensitivity to the components  
of this product, as patients with allergic histories to the  
components of this formulation must present examinations  
to a specialist before using this product;  
• Always advise your patient to return to the clinic to  
monitor the treatment always after one week (7 days), for  
evaluations and to ensure the effectiveness of the results of  
tooth whitening until the end of treatment;  
• Always keep away from heat and/or sunlight. DSP White  
Endo Gel can be stored at room temperature;  
• Non-injectable product. Do not reuse the packaging.

SIDE EFFECTS:  
• Intense internal pressure may occur in the pulp chamber  
due to the decomposition of the peroxide.  
• If you notice that the element is lightening very quickly.  
The professional should assess the need to discontinue  
treatment early.  
• Avoid the imposition of forces on the tooth that is being  
whitened, due to the risk of fracture due to the fragility of the  
tooth during the whitening treatment.  
• Avoid coloring in foods and drinks.  
Discontinue use of the product in case of allergic reactions.

CONSERVATION AND STORAGE:  
• Keep the product in its packaging always closed.  
• Protect from the incidence of direct sunlight.  
• Store the product at temperatures from 0°C to 30°C.  
• If they find malfunctions, request the replacement of the  
material.  
Note: When refrigerated at very low temperatures, micro  
crystallizations of Carbamide Peroxide may occur due to the  
high concentration of the active ingredient therein. It is  
recommended that the product be removed from  
refrigeration at least 30 minutes before application, leaving  
it at room temperature.

DATE OF MANUFACTURE: PACKING  
EXPIRATION DATE: 2 YEARS AFTER THE DATE OF MANUFACTURE.

WARNING:  
Respect the expiration date. Never use the product if it  
has expired. Keep out of reach of children. For dental use  
only. Before using observe the appearance of the product,  
if you notice any anomalies, contact the manufacturer  
or a professional immediately. The manufacturer is not  
responsible for damages caused by other uses or by  
incorrect handling. This material has been manufactured  
for dental use only and must be handled according to the  
instructions for use. It is the responsibility of the user to prove,  
before the use of this product, that this material is compatible  
and suitable for the desired use, especially when this use is  
not indicated in these instructions for use. Data descriptions  
do not constitute any kind of guarantee and, therefore, do  
not have any link. Do not reuse empty packages. Do not  
reuse empty packaging for other purposes. To dispose of the  
product, follow local legislation.

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