# DSP *white*

ENDO PERBORATE

EN | ENGLISH

PT | PORTUGUÊS

ES | ESPAÑOL

DSP  
*white*

# DSP white

ENDO PERBORATE

ENGLISH

To access the IFU go to  
dspbiomedical.com/ifu  
with code: 0.5002

TRADE NAME: DSP White Endo Perborate  
TECHNICAL NAME: Dental Whitening Agents

Please read all information of this instruction for use carefully  
before applying the product.

DENTAL WHITENER FOR CLINICAL USE.  
FOR HOME APPLICATION UNDER THE SUPERVISION OF A DENTAL  
SURGEON  
SALE UNDER PRESCRIPTION (ACCORDING TO RDC NO. 6 OF  
2015)

Keep for consultation until the total consumption of the  
product.  
Keep this instruction until there is no more interaction of the  
product with your patient.

#### CONTENT:

- 1 bottle of Sodium Perborate 10g;
- 1 bottle of Hydrogen Peroxide Solution 20% 10g;
- 1 powder measure;
- 1 plate for blending;
- 1 instruction manual for the professional.

#### PRODUCT DESCRIPTION:

DSP White Endo Sodium Perborate is a whitening system  
based on sodium perborate and 20% hydrogen peroxide,  
for professional use. Sodium perborate is a powdered  
substance that in contact with water decomposes into  
sodium metaborate, hydrogen peroxide and oxygen. It is  
used in combination with hydrogen peroxide for whitening in  
non-vital teeth.

To perform the tooth restoration after whitening, it is  
recommended to wait 7 to 15 days, because in this interval  
there is the stabilization of the final color and the elimination  
of any traces of oxygen derivatives that can interfere with the  
polymerization of the adhesive systems and the composite  
resins used in the restorative treatment.

The dimensions of the color (hue, chroma and value) are  
influenced by the presence of the dental pulp. When it ceases  
to exist, for example, in cases of pulp necrosis, a change in  
the color and brightness of the tooth can occur, usually with  
a dark tone and with shades that can vary between gray,  
green or blue.

The main causes of color changes in the crown of extirpated  
teeth are: Pulp hemorrhage caused by trauma; hemorrhage  
after pulp extirpation; insufficient coronary opening;  
decomposition of the coronary pulp; medicines and  
substances used in the treatment of the canal; permanent  
and temporary restorative materials; pulp calcification.

The tooth darkening process occurs due to the formation of  
chemically stable structures, responsible for the progressive  
installation of stains on the dental crown. The basic whitening  
process involves oxidation, which consists of a chemical  
process where organic materials are converted into carbon  
dioxide and water.

Pigments are composed of a large amount of carbon  
molecules. These are broken down and converted into  
intermediate compounds (smaller chains) that are lighter.  
This chemical reaction changes the type, number, and  
relative position of the atoms that make up these molecules.  
During the course of the whitening, the carbon chains are  
transformed into carbon dioxide and water, being gradually  
released together with the nascent oxygen.

#### PRODUCT INDICATION:

DSP White Endo Perborate is a whitening system based  
on Sodium Perborate and 20% Hydrogen Peroxide for  
professional use, indicated for the whitening of non-vitalized  
teeth. DSP White Endo Perborate should be administered  
inside the pulp chamber of the tooth to be whitened, and  
should remain for a period to oxidize the existing pigments.

#### COMPOSITION:

Phase 1 Powder: Sodium perborate tetrahydrate;  
Phase 2 Liquid: Concentrated Hydrogen Peroxide Solution  
20%  
After blending of phases: Hydrogen Peroxide 15% - 25%,

Thickeners and Deionized Water.

#### INSTRUCTIONS FOR USE:

1st step

Verify through clinical and radiographic examination the  
level of excellence of endodontic filling;

2nd step

Perform prophylaxis;

3rd step

Recording the current color for later comparison;

4th step

Absolute isolation to avoid contact of the product with  
adjacent soft tissues;

5th step

Clean the pulp chamber;

6th Step

Based on the amelocementary limit (verified on the x-ray),  
remove 3-4mm of obturator material beyond this anatomical  
limit and create a cervical plug of approximately 2mm (it is  
recommended to use chemically activated glass ionomer  
or other sealer of your choice). This type of restoration aims  
to prevent the permeation of excess nascent oxygen to the  
dentin/endodontic filling interface and to the periodontium,  
minimizing the chances of pericementitis and external  
cervical resorption, respectively.

7th Step

Perform the acid conditioning of the entire pulp chamber  
and the superficial cavity angle of the coronary opening,  
wash in abundance and dry. This procedure aims to remove  
the Smear Layer that forms during the instrumentation  
of the cavity and facilitates the action of DSP White Endo  
Perborate; the conditioning of the surface cavity facilitates  
the subsequent retention of the temporary restoration.

8th Step

Mix 1 measures of DSP White Endo Perborate (powder phase)  
with 1 drop of DSP White Endo Perborate (liquid phase), until it  
forms a paste of firm consistency, and insert this paste inside  
the pulp chamber;

9th Step

Seal the cavity with a temporary restorer and reassess the  
case after 5 to 7 days;

Step 10

If necessary, repeat the application to achieve the desired  
effect, re-evaluating after 3 to 4 days. It is not necessary to  
repeat the acid conditioning.

Step 11

After completing the whitening treatment, it is recommended  
to wait at least one week to perform the definitive restoration.  
This time for restoration is suggested due to the release of  
oxygen, as it may interfere with the polymerization of the  
composite resin and impair its bonding with the whitened  
tooth.

#### PRECAUTIONS AND CONTRAINDICATIONS:

- This is a product that must always be indicated and  
supervised by a dentist;
- The product should not be used by pregnant and/or  
lactating women and in patients under 15 years old;
- Avoid the purposeful and excessive ingestion of the product;
- Do not use in patients with any sensitivity to the components  
of this product, as patients with allergic histories to the  
components of this formulation must present examinations  
to a specialist before using this product;
- Always advise your patient to return to the clinic to  
monitor the treatment always after one week (7 days), for  
evaluations and to ensure the effectiveness of the results of  
tooth whitening until the end of treatment;
- Always keep away from heat and/or sunlight. DSP White  
Endo Perborate can be stored at room temperature;
- Non-injectable product. Do not reuse the packaging.

#### SIDE EFFECTS:

- Intense internal pressure may occur in the pulp chamber  
due to the decomposition of the peroxide.
  - If you notice that the element is lightening very quickly.  
The professional should assess the need to discontinue  
treatment early.
  - Avoid the imposition of forces on the tooth that is being  
whitened, due to the risk of fracture due to the fragility of the  
tooth during the whitening treatment.
  - Avoid coloring in foods and drinks.
- Discontinue use of the product in case of allergic reactions.

#### CONSERVATION AND STORAGE:

- Keep the product in its packaging always closed.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0°C to 30°C.
- If they find malfunctions, request the replacement of the  
material.

DATE OF MANUFACTURE: PACKING

EXPIRATION DATE: 2 YEARS AFTER THE DATE OF MANUFACTURE.

#### WARNING:

Respect the expiration date. Never use the product if it  
has expired. Keep out of reach of children. For dental use  
only. Before using observe the appearance of the product,  
if you notice any anomalies, contact the manufacturer  
or a professional immediately. The manufacturer is not  
responsible for damages caused by other uses or by  
incorrect handling. This material has been manufactured

for dental use only and must be handled according to the  
instructions for use. It is the responsibility of the user to prove,  
before the use of this product, that this material is compatible  
and suitable for the desired use, especially when this use is  
not indicated in these instructions for use. Data descriptions  
do not constitute any kind of guarantee and, therefore, do  
not have any link. Do not reuse empty packages. Do not  
reuse empty packaging for other purposes. To dispose of the  
product, follow local legislation.

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