DSP *white*

PERSONAL



EN | ENGLISH

PT | PORTUGUÊS

ES | ESPAÑOL

DSP
white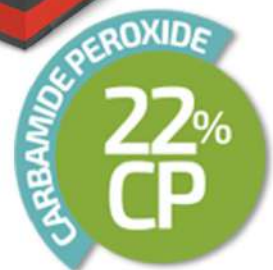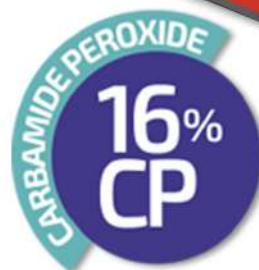

DSP white

PERSONAL

ENGLISH

To access the IFU go to
dspbiomedical.com/ifu
with code: 0.5003

TRADE NAME: DSP White Personal
TECHNICAL NAME: Dental Whitening Agents

Please read all information of this instruction for use carefully before applying the product.

DENTAL WHITENER FOR HOME USE.
FOR HOME APPLICATION UNDER THE SUPERVISION OF A DENTAL SURGEON
SALE UNDER PRESCRIPTION (ACCORDING TO RDC NO. 6 OF 2015)

Keep for consultation until the total consumption of the product.
Keep this instruction until there is no more interaction of the product with your patient.

CONTENT:
DSP White Personal Kit 10%, 16% and 22%:
• 5 whitening gel syringes containing 3g with 5 applicator tips,
• 2 EVA plates 1mm thick,
• 1 tray holder case,
• 1 instruction manual for the professional and patient.

Mini DSP White Personal Kit 10%, 16% and 22%:
• 3 whitening gel syringes containing 3g with 3 applicator tips,
• 1 instruction manual for the professional and patient.

BULK SET DSP White Personal 10%, 16% and 22%:
• 40 syringes with 3g and applicator tips.
• 1 patient instruction manual.

PRODUCT DESCRIPTION:

DSP White Personal was developed based on products already known worldwide and highly recommended for whitening of vital teeth. Its formula combines the high viscosity of the Carbomer gel that does not leak from the tray, like other whiteners. With the active principle of Carbamide Peroxide to obtain an effective dental whitening. It has the pH of the neutral gel so as not to cause the decalcification of the enamel and dentin. Its formulation also contains the powerful humectant (Glycerol) and a high water content that helps prevent dental dehydration. DSP White Personal has a crystalline appearance and with a unique blend of Potassium Nitrate and Sodium Fluoride as desensitizing agents that reduce tooth sensitivity during whitening to the maximum. They are available in concentrations of 10%, 16% and 22%. They can be used during the night (while the patient is sleeping) or during the day. At concentrations of 10% and 16% it is recommended to use for a period of 3 to 4 hours daily. For the concentration of 22%, it is recommended to use for 1 hour daily.

In normal cases the treatment lasts from 10 to 14 days. The frequency or time using the gel may vary depending on the effects of the patient's sensitivity.

The DSP White Personal gel based on Carbamide Peroxide contains Potassium Nitrate and 0.25% Fluoride ions. It has also been proven that DSP White Personal reduces the susceptibility of cavities, reduces the sensitivity, improving the microhardness of the enamel.

The patient will quickly notice the results of the whitening becoming visible after the 3rd day of use of the whitening gel, which motivates him to continue the treatment. Treatment lasts approximately 15 days when the whitening gel is used daily. Vital teeth whitened with DSP White Personal remain clear for years. DSP White Personal whitening gel is available in various concentrations to meet all your needs.

PRODUCT INDICATION:

DSP White Personal is indicated for whitening of vital and non-vital teeth based on the dental whitening technique of home use supervised by the dentist. DSP White Personal is effective in the removal of stains due to congenital, systemic, pharmacological or traumatic factors, in addition to stains due to aging. They present good results against stains resulting from fluorosis, especially the brown stain. It

is also indicated against first-degree and second-degree tetracycline stains.

HOW TO CHOOSE THE CONCENTRATION:

10% - Normal pigmentation due to age, coffee or smoke or when the patient's sensitivity is intense.
16% and 22% - In cases of more pronounced pigmentation and greater difficulty in removal or when visible results are desired in a short time.

COMPOSITION:

Carbamide Peroxide, Carbomer, Potassium Nitrate, Sodium Fluoride, Humectant (Glycerol), neutralizers and Deionized Water.

DENTAL SURGEON USAGE INSTRUCTIONS:

1st step

Empty the casting with plaster. Moldings with alginate should be leaked quickly to ensure accuracy. Cutting the tray is facilitated when using smaller amounts of gypsum.

2nd step

With a Vacuum Forming, heat the EVA plate until it bends about an inch and a half. Adapt the plastic to the model. Cool and remove the template.

3rd step

Use the small scissors to accurately cut the tray, from 0.20 to 0.30 mm from the gingival margin. If possible smooth the edges of the tray to avoid trauma.

4th step

Load the tray, applying a drop or fillet of gel to each tooth, only from the vestibular side.

5th step

Remove excess gel with clean fingers, cotton, soft brush, handkerchief, and plenty of water. Follow the dentist's instructions.

6th Step

Use instruction follow in the patient instruction manual.

USAGE INSTRUCTIONS – PATIENT:

1st step

Put some gel in the tray compartment on each tooth that should be treated.

2nd step

Place the tray in the mouth so that the gel overlaps the teeth.

3rd step


Remove excess with a sanitary wipe or a dry cotton swab.

4th step

After treatment, remove the tray. Rinse the tray before storing it and before use. Store the gel cool. Avoid prolonged exposure of the syringe to light. Properly dispose of empty syringes.

5th step

Brushing your teeth and doing oral hygiene.

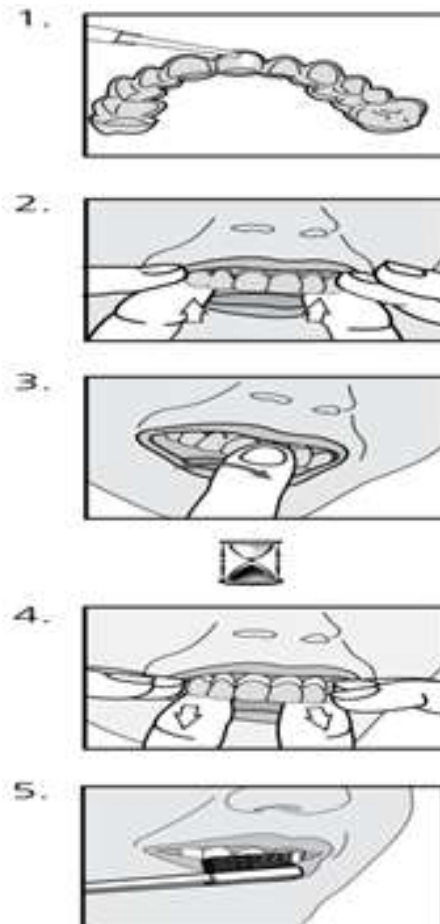PRECAUTIONS AND CONTRAINDICATIONS:

- This is a product that must always be indicated and supervised by a dentist;
- Before starting treatment, inform your patient about the possibility of dental sensitivity;
- The product should not be used by pregnant and/or lactating women and in patients under 15 years old;
- Avoid the purposeful and excessive ingestion of the product;
- Do not use in patients with any sensitivity to the components of this product, as patients with allergic histories to the components of this formulation must present examinations to a specialist before using this product;
- During the treatment, the patient should avoid the ingestion of beverages, fruits and acid products, thus avoiding sensitivity;
- Give the patient the brochure with his instructions and emphasize the importance of reading it;
- Always advise your patient to return to the clinic to monitor the treatment always after one week (7 days), for evaluations and to ensure the effectiveness of the results of tooth whitening until the end of treatment;
- If any problems occur, stop the treatment and notify your dentist;
- Always keep away from heat and/or sunlight. DSP White Personal can be stored at room temperature;
- If a large volume of product is accidentally swallowed, seek medical help;
- Non-injectable product. Do not reuse the packaging.

SIDE EFFECTS:

- Ingestion of large amounts of the product may cause nausea.
 - Before starting treatment, inform your patient about the possibility of dental sensitivity. It is usually common to have dental sensitivity during the treatment of tooth whitening. In this case, if sensitivity persists, treat the teeth with desensitizing gel DSP Desensitizer 0.2% NaF indicated for moderate to low sensitivity or desensitizing gel DSP Desensitizer 2% NaF indicated for moderate to high sensitivity.
 - Excessive gel in the tray can leave a slight irritation in the gingival tissue, so always remove the excess according to the patient's instruction manual.
- Discontinue use of the product in case of allergic reactions.

CONSERVATION AND STORAGE:

- Keep the product in its packaging always closed.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0°C to 30°C.
- If they find malfunctions, request the replacement of the material.

DATE OF MANUFACTURE: PACKING

EXPIRATION DATE: 2 YEARS AFTER THE DATE OF MANUFACTURE.

WARNING:

Respect the expiration date. Never use the product if it has expired. Keep out of reach of children. For dental use only. Before using observe the appearance of the product, if you notice any anomalies, contact the manufacturer or a professional immediately. The manufacturer is not responsible for damages caused by other uses or by incorrect handling. This material has been manufactured for dental use only and must be handled according to the instructions for use. It is the responsibility of the user to prove, before the use of this product, that this material is compatible and suitable for the desired use, especially when this use is not indicated in these instructions for use. Data descriptions do not constitute any kind of guarantee and, therefore, do not have any link. Do not reuse empty packages. Do not reuse empty packaging for other purposes. To dispose of the product, follow local legislation.

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PROIBIDO REPROCESSAR

Prohibited Reprocess
Proibido Reprocessar
Prohibido reprocessar

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