



DSP DAM

MANUFACTURED AND DISTRIBUTED BY:



DSP Industrial LTDA | CNPJ: 03.960.018/0001-23

Rua Marechal Floriano Peixoto, 303 Ouro Verde II - Campo Largo - PR 83.606-290
- SAC 0800.600.88.66

Resp. Técnico: Célio G. Netzel
CREA-PR: 25412/D

Anvisa Registration: 80116980034

FOR PURCHASES, QUESTIONS, COMPLIMENTS, AND COMPLAINTS: 0800 600 8866

INSTRUCTIONS FOR USE | english

TRADE NAME: DSP Dam

TECHNICAL NAME: Light Cured Gingival Barrier

CÓDIGO: 0.1300

Please read all information of this instruction for use carefully before applying the product.

INSTRUCTIONS FOR USE

PROFESSIONAL USE ONLY

Keep for consultation until the total consumption of the product. Keep this instruction until there is no more interaction of the product with your patient.

CONTENT:

- 1 syringe with 2g and 2 applicator tips,
- 1 instruction manual for the professional.

PRODUCT DESCRIPTION:

DSP Dam is a light-cured resin that functions as a light-cured that replaces the rubber sheet when gingival isolation is relative. The main objective of the product is to protect soft tissues from contact with hydrogen peroxide (whitening gel).

When in contact with the gum, hydrogen peroxide or whitening gel can damage the tissues and generate great discomfort, thus interfering at the end of the whitening. DSP Dam acts in order to avoid this discomfort, ensuring a better performance in the treatment of tooth whitening.

PRODUCT INDICATION:

It is indicated as a substitute for the rubber sheet for the protection of gingival tissue and in cases of risk of irritation due to contact with products used in the clinical procedure, such as tooth whitening and as an adjunct to absolute isolation for better sealing of a cavity preparation or endodontic treatment.

DSP DAM

COMPOSITION:

Dimethacrylate Groups; Photoinitiators; pigments and colorants.

INSTRUCTIONS FOR USE:

1st step

Dry the gums and teeth corresponding to the region you want to protect.

2nd step

Apply with the aid of the applicator tip a layer approximately 1 mm wide of the product on the gums including the previously dried papillae. Attach the applicator tip to the syringe and apply a layer approximately 1 mm wide on the gum including the previously dried papillae.

3rd step

Polymerize the resin using 20 to 30 seconds of light from a LED or halogen light curing device, for groups of 3 to 4 teeth. Proceed to polymerization soon after applying the resin to prevent any possible irritation.

4th step

Check that the product covers the entire surface of the gum and that it is completely polymerized. Avoid testing it by forcing or trying to lift it, this can cause infiltration of the irritating substance.

5th step

To remove the product just detach it forward, until it easily loosens in full or in large pieces.

PRECAUTIONS AND CONTRAINDICATIONS:

Avoid applying the product to the gums of patients who have recently undergone some type of oral surgery or who have any type of gingival irritation. Incomplete polymerization of the product or excessive time can lead to mild gingival irritation in some cases.

SIDE EFFECTS:

The product has methacrylate monomers that can cause allergic reactions or mild irritations to patients sensitive to these substances. Discontinue use of the product in case of allergic reactions.

CONSERVATION AND STORAGE:






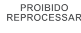








- Keep the product in its packaging always closed.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0°C to 30°C.
- If they find malfunctions, request the replacement of the material.

DATE OF MANUFACTURE: PACKING

EXPIRATION DATE: 2 YEARS AFTER THE DATE OF MANUFACTURE.

WARNING:

Respect the expiration date. Never use the product if it has expired. Keep out of reach of children. For dental use only. Before using observe the appearance of the product, if you notice any anomalies, contact the manufacturer or a professional immediately. The manufacturer is not responsible for damages caused by other uses or by incorrect handling. This material has been manufactured for dental use only and must be handled according to the instructions for use. It is the responsibility of the user to prove, before the use of this product, that this material is compatible and suitable for the desired use, especially when this use is not indicated in these instructions for use. Data descriptions do not constitute any kind of guarantee and, therefore, do not have any link. Do not reuse empty packages. Do not reuse empty packaging for other purposes. To dispose of the product, follow local legislation.

	Code of Product
	Number of the lot
	Manufacturer
	Date of manufacturing
	Representative in the community european
	Prohibited Reprocess
	Date of Durability Minimum
	Do not reuse
	Do not use if the package it's damaged
	Keep Dry
	Keep away from light
	Notification demanded by FDA for Business In the USA
	Reference to information join or attached
	Irritante