



DSP DESENSITIZER 0,2% NAF

MANUFACTURED AND DISTRIBUTED BY:



DSP Industrial LTDA | CNPJ: 03.960.018/0001-23

Rua Marechal Floriano Peixoto, 303 Ouro Verde II - Campo Largo - PR 83.606-290
- SAC 0800.600.88.66

Resp. Técnico: Célio G. Netzel
CREA-PR: 25412/D

Anvisa Registration: 80116980031
FOR PURCHASES, QUESTIONS, COMPLIMENTS, AND
COMPLAINTS: 0800 600 8866

INSTRUCTIONS FOR USE | english

TRADE NAME: DSP Desensitizer 0,2% naf

TECHNICAL NAME: Substance for Treatment of Dental Hypersensitivity

CÓDIGO: 0.1400

Please read all information of this instruction for use carefully before applying the product.

INSTRUCTIONS FOR USE

PROFESSIONAL USE ONLY

Keep for consultation until the total consumption of the product.

Keep this instruction until there is no more interaction of the product with your patient.

CONTENT:

- 1 syringe with 3g and 1 applicator tip,
- 1 instruction manual for the professional and patient.

PRODUCT DESCRIPTION:

DSP Desensitizer 0.2% is a neutral pH low-viscosity topical desensitizing gel with dual desensitizing action based on Potassium Nitrate (neural action) and Sodium Fluoride (occlusion action) that does not interfere with the speed of tooth whitening. It can be used before the first whitening session, between sessions and after the dental whitening sessions. The patient can take Desensitizer 0.2% for home treatment. The product should never be mixed with the tooth whitening gel.

PRODUCT INDICATION:

DSP Desensitizer 0.2% is indicated in the treatment of teeth with moderate to low sensitivity and has been developed to relieve tooth sensitivity triggered by thermal and chemical agents. It is also indicated in the treatment of sensitivity that can be caused by the treatment of tooth whitening.

DSP DESENSITIZER 0,2% NAF

COMPOSITION:

6% Potassium Nitrate, 0.2% Sodium Fluoride, Humectant (Glycerol), Neutralizers, Thickeners and Deionized Water.

INSTRUCTIONS FOR USE:

1st step

Clean the dental surface and eliminate any deposit that may prevent application of the product to the teeth.

2nd step

Attach the tip to the syringe and apply desensitizing gel DSP Desensitizer 0.2% evenly over the teeth for at least 10 minutes. The tooth whitening tray can be used by applying the desensitizing gel. Before this procedure, remove the whitening gel from the tray and clean the desensitizing gel. Fit the tray to the teeth and press lightly to enclose the dental surface with the gel.

3rd step

After exceeding the application time, remove the desensitizing gel from the dental surface with the help of clean fingers, cotton, soft brush, sanitary wipe and plenty of water, preventing the patient from swallowing the desensitizing gel.

4th step

If it is necessary to repeat the process of applying the desensitizing gel every 2 or 3 hours until the symptoms are relieved. In case of tooth whitening, follow the treatment time.

PRECAUTIONS AND CONTRAINDICATIONS:

Avoid the purposeful and excessive ingestion of the product;

Do not use in patients with any sensitivity to the components of this product;

If any problems occur, stop the treatment and notify your dentist;

Always keep away from heat and/or sunlight. DSP Desensitizer 0.2% can be stored at room temperature;

If a large volume of product is accidentally swallowed, seek medical help;

Non-injectable product; Do not reuse the packaging.
If they find malfunctions, request the replacement of the material.

SIDE EFFECTS:

Ingestion of large amounts of the product may cause nausea.
Discontinue use of the product in case of allergic reactions.

CONSERVATION AND STORAGE:






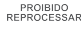







- Keep the product in its packaging always closed.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0°C to 30°C.

DATE OF MANUFACTURE: PACKING

EXPIRATION DATE: 2 YEARS AFTER THE DATE OF MANUFACTURE.

WARNING:

Respect the expiration date. Never use the product if it has expired. Keep out of reach of children. For dental use only. Before using observe the appearance of the product, if you notice any anomalies, contact the manufacturer or a professional immediately. The manufacturer is not responsible for damages caused by other uses or by incorrect handling. This material has been manufactured for dental use only and must be handled according to the instructions for use. It is the responsibility of the user to prove, before the use of this product, that this material is compatible and suitable for the desired use, especially when this use is not indicated in these instructions for use. Data descriptions do not constitute any kind of guarantee and, therefore, do not have any link. Do not reuse empty packages. Do not reuse empty packaging for other purposes. To dispose of the product, follow local legislation.

	Code of Product
	Number of the lot
	Manufacturer
	Date of manufacturing
	Representative in the community european
	Prohibited Reprocess
	Date of Durability Minimum
	Do not reuse
	Do not use if the package it's damaged
	Keep Dry
	Keep away from light
	Notification demanded by FDA for Business In the USA
	Reference to information join or attached