



## DSP POLISHING PASTE AL – ONE & TWO

### MANUFACTURED AND DISTRIBUTED BY:

---



DSP Industrial LTDA | CNPJ: 03.960.018/ 0001-23

Rua Marechal Floriano Peixoto, 303 Ouro Verde II – Campo Largo – PR 83.606-290  
- SAC 0800.600.88.66

Resp. Técnico: Célio G. Netzel  
CREA-PR: 25412/D

Anvisa Registration: 80116980033

FOR PURCHASES, QUESTIONS, COMPLIMENTS, AND COMPLAINTS: 0800 600 8866

## INSTRUCTIONS FOR USE | english

TRADE NAME: DSP Polishing Paste AI – One & Two

TECHNICAL NAME: Dental Prophylaxis and Polishing Pastes

CÓDIGO: 0.4000

Please read all information of this instruction for use carefully before applying the product.

### INSTRUCTIONS FOR USE

#### PROFESSIONAL USE ONLY

Keep for consultation until the total consumption of the product.

Keep this instruction until there is no more interaction of the product with your patient.

#### CONTENT:

- 1 syringe with 4.0g DSP Polishing Paste AI – One – 40-50 microns with 1 applicator tip;
- 1 syringe with 4.0g DSP Polishing Paste AI – Two – 8-10 microns with 1 applicator tip;
- 1 instruction manual for the professional and patient.

#### PRODUCT DESCRIPTION:

Polishing paste based on granulation Aluminum Oxide – 8.3 to 46.5 microns, prepared for the initial surface finish. It offers excellent results in the initial finishing and preliminary polishing of resins and other restorative materials. For professional use only. DSP Polishing Paste AI – One & Two paste set combines in its formulation the lubricating and abrasive properties of high hardness. The DSP Polishing Paste AI – One paste is medium-grained 40-50 microns and is indicated for initial surface finishing. DSP Polishing Paste AI – Two is finegrained 8-10 microns is indicated for the preliminary polishing to the final polishing. Its formulation was developed for exclusive use in a dental clinic and contains a medium of the paste composed of water-soluble ingredients that provides ease in lubrication during the polishing

## DSP POLISHING PASTE AI – ONE & TWO

process and does not suffer from the action of heat. It has medium viscosity that facilitates at the time of polishing.

#### PRODUCT INDICATION:

DSP Polishing Paste AI – One & Two is indicated for use in conjunction with felt discs or velvety discs, which enhances its action in polishing for the finishing and pre-polishing of resins and other restorative materials.

#### COMPOSITION:

Aluminum Oxide, lubricating medium, thickeners and aroma.

#### INSTRUCTIONS FOR USE:

##### 1st step

Prepare the surface to be polished using the medium granulation paste (One) on a felt disc leaving it smooth and in the desired final shape. This step must be strictly observed because, generally, obtaining gloss has a strong dependence on the initial surface condition.

##### 2nd step

Using the fine-grained paste (Two), fit the felt disc to the end of the mandrel. Dispense the paste onto the disc or directly onto the tooth or restorative material. Apply the product at a low speed, exerting slight pressure, with intermittent touches using a circular motion.

##### 3rd step

If you have not obtained the desired result, repeat the procedure.

#### PRECAUTIONS AND CONTRAINDICATIONS:

Use low speed, exerting slight pressure on the felt discs, with intermittent touches using a circular motion. To avoid excessive heating of the dental surface. During application of the product it is recommended to wear safety glasses by the professional. Do not use in patients with any sensitivity to the components of this product. Discontinue use of the product in case of allergic reactions.

**CONSERVATION AND STORAGE:**














- Keep the product in its packaging always closed.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0°C to 30°C.
- If they find malfunctions, request the replacement of the material.

DATE OF MANUFACTURE: PACKING

EXPIRATION DATE: 2 YEARS AFTER THE DATE OF MANUFACTURE.

**WARNING:**

Respect the expiration date. Never use the product if it has expired. Keep out of reach of children. For dental use only. Before using observe the appearance of the product, if you notice any anomalies, contact the manufacturer or a professional immediately. The manufacturer is not responsible for damage caused by other uses or by incorrect handling. This material has been manufactured for dental use only and must be handled according to the instructions for use. It is the responsibility of the user to prove, before the use of this product, that this material is compatible and suitable for the desired use, especially when this use is not indicated in these instructions for use. Data descriptions do not constitute any kind of guarantee and, therefore, do not have any link. Do not reuse empty packages. Do not reuse empty packaging for other purposes. To dispose of the product, follow local legislation.

	Code of Product
	Number of the lot
	Manufacturer
	Date of manufacturing
	Representative in the community european
	Prohibited Reprocess
	Date of Durability Minimum
	Do not reuse
	Do not use if the package it's damaged
	Keep Dry
	Keep away from light
	Notification demanded by FDA for Business In the USA
	Reference to information join or attached